


The Monarch Mark 1-A

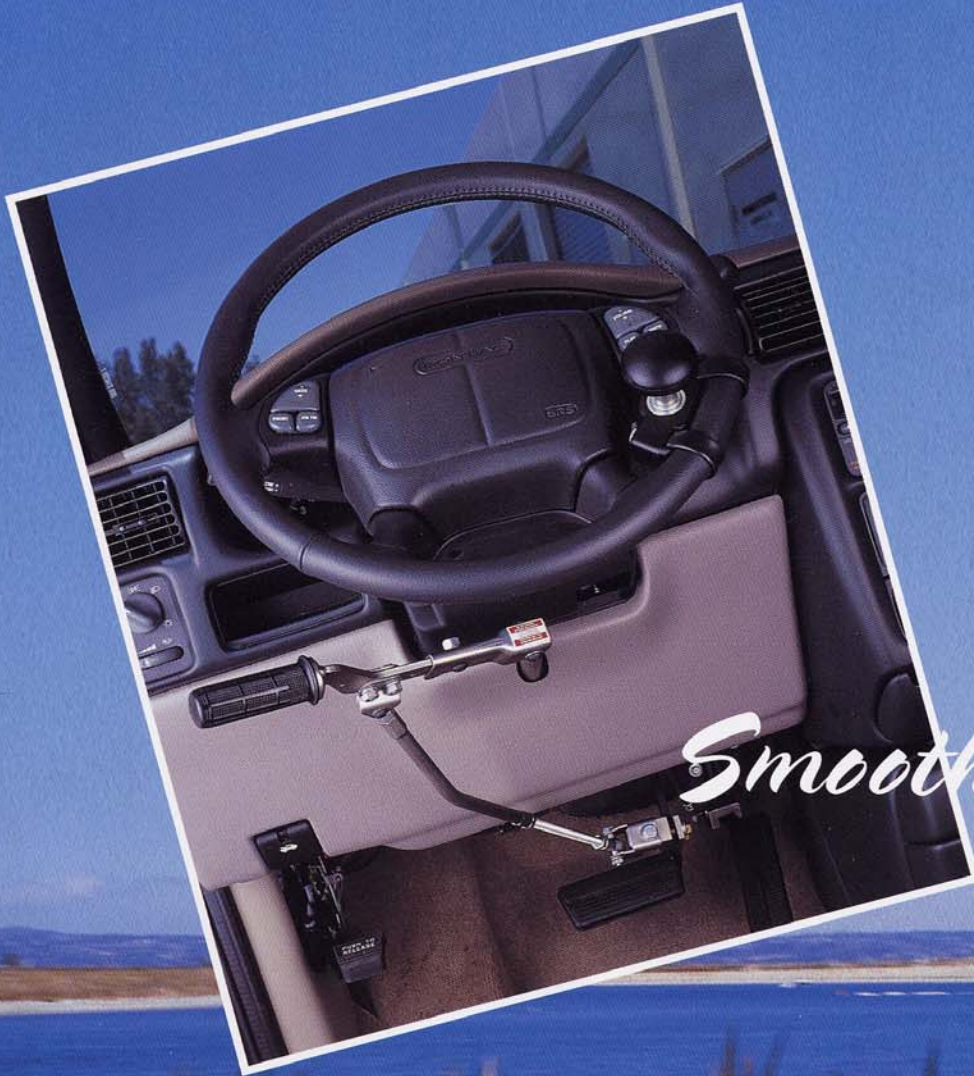

Smooth Operator



Manufacturing and Production Services Corporation



The Monarch Mark 1-A

The Hand Control recommended most often by rehabilitation driving specialists.

MPS's Monarch 1-A is the right choice in mechanical hand controls. It is a *smooth operator*, offering the most popular and most recommended method of operation. This *smooth operating* control allows for miles and miles of relaxing trouble-free driving.

The brake is applied by pushing the operating handle directly towards the brake pedal. The accelerator is activated by moving the operating handle toward the driver's seat, at a right angle to the brake movement. These movements are the least fatiguing. The brake and accelerator can be used separately or simultaneously.

Installation of MPS's products require a minimum amount of modifications to the vehicle. All hand controls and accessories may be used for either right or left hand activation. The hand control does require automatic transmission, and must only be installed on vehicles with power steering and power brakes.


Optional Accessories

Brake Only

The brake only control is available for persons with sufficient foot motion and strength to operate the accelerator but not the brake. Should an accelerator control become necessary, the needed components can be added to the existing control.

Toggle Type Dimmer Switch and Horn Button

A toggle type dimmer switch and separate Horn Button are available for use with "Monarch Mark 1-A" or brake only hand control.

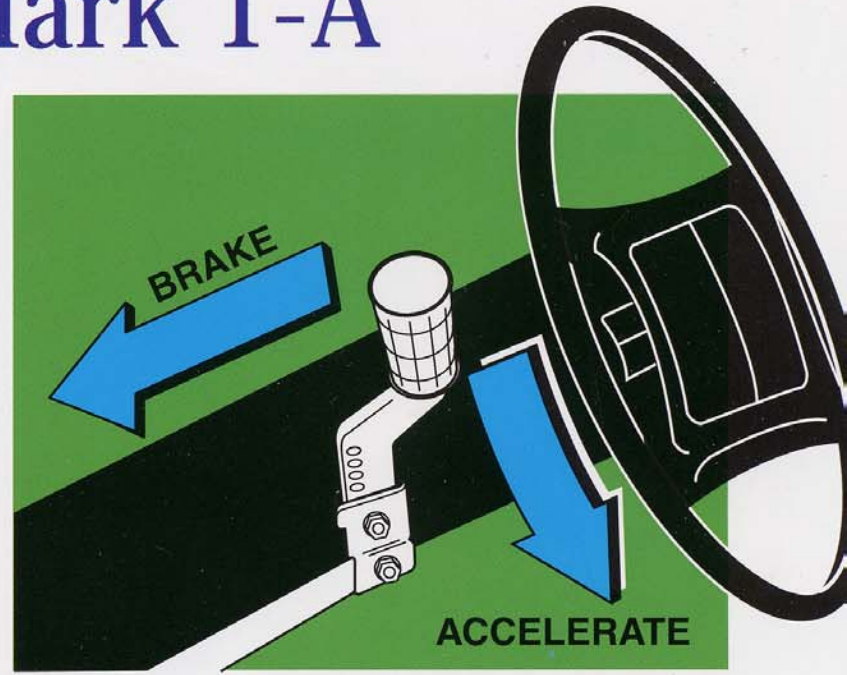

Features:

- Large comfortable hand grip
- Operating handle adjustable for each drivers optimal comfort
- Adjustable mounting assembly to provide maximum leg room and proper alignment
- Adjustable brake attachment rod to match each application
- Universal accelerator attachment to provide proper accelerator leverage
- An advanced accelerator return spring that allows the accelerator pedal to move freely without the movement of the operating handle when driven without the hand control by others
- Retains tilt steering wheel option for most cars with this feature
- Easily reinstalled to a new or different vehicle

Steering Devices




Quad Fork



Spinner Knob



Palm Spinner

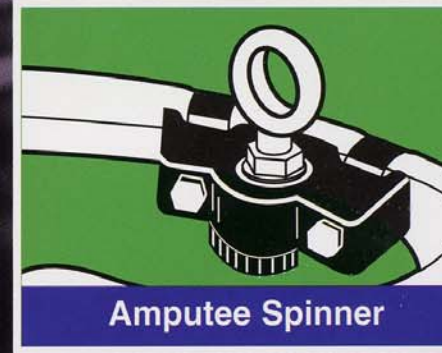


Amputee Spinner



Tri Pin
(single pin also available)

These devices are available to assist your handling of the steering wheel. All steering devices are easily removed for other drivers.

Other Driving Aids

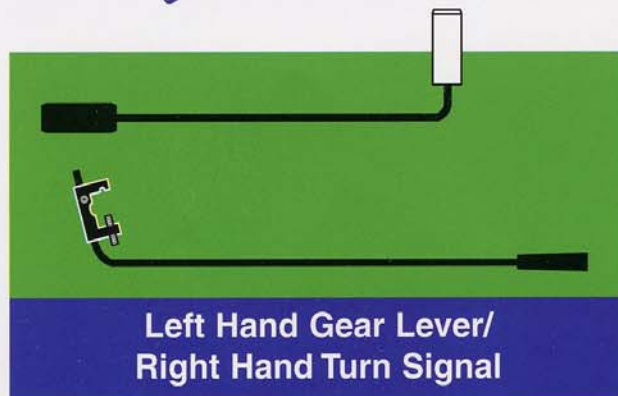

Left Hand Gear Lever/
Right Hand Turn Signal



Parking Brake
Extension



Left Foot Gas Pedal with Quick Release Base

This feature is available for drivers with impaired right foot strength and control. It can be installed quickly through our dealer network and is removable for other drivers with the press of a button.

Serial Number

Provided with each driving aid is a return registration card to be completed and returned with the exact items purchased and the serial number of the unit. This is your guarantee that accurate information regarding the appliance can be obtained from the manufacturer when requested.

30 Day Return Guarantee

Any driving equipment may be returned within thirty (30) days from the date of purchase by the customer for a full price refund except for any item considered custom or special. This refund includes the price of individual item(s) only and is not applicable to service charges for installation or any freight or handling charges. No obligation or cost whatsoever will be assumed for any modifications or damage to any motor vehicle in which the equipment was installed.

Warranty

MPS Driving Aids are guaranteed to be free from defective parts or components for five years from the date of purchase to the original owner, and for the original vehicle for which the equipment was purchased. Electrical components or accessories are guaranteed for ninety (90) days from the date of purchase. Any part of the component found to be defective during the warranty period may be returned.

A replacement part will be shipped immediately. The manufacturer assumes obligation and responsibility for any defects or damage caused in manufacturing only. This warranty is in lieu of all other guarantees or warranties, expressed or implied. The returned registration card qualifies each driving aid is warranted. The registration card should be completed with the registered serial number of our unit allowing MPS to provide accurate information regarding your driving aids when requested.